

## Hygiene Pro Air

The Hygiene Pro is a relatively trouble free motor but a few things to be aware of.


### **Nosecone Coming Off**

The blue sheath is screwed onto the motor and over a period of time can become unscrewed. Even if the sheath is slightly unscrewed it can prevent the nosecone from being seated properly.

The sheath is cross threaded to prevent it from coming unscrewed while removing the motor but can loosen while putting the motor on the tubing. If a customer complains of the nosecone not seating properly make sure the sheath is screwed down into place.

### ***Tip:***

When placing the hygiene Pro Air motor onto the tubing grab the lower end of the motor where the blue and silver meet.

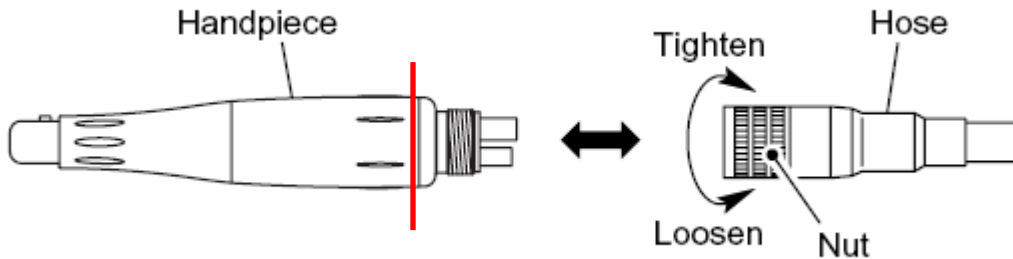

### **Prophy Angle Sounds Loud**

If the prophy angle head is vibrating or making loud noises it may need a slight adjustment on the nosecone. Simply pull up on the prophy angle so there is a 1-2mm gap between the nosecone and the prophy should do the trick and eliminate the vibration.

