

# solutions™

DEVELOPED BY BRASSELER USA

## DIALITE® LD ADJUSTING, FINISHING & POLISHING SYSTEM FOR LITHIUM DISILICATE



B-3798B-REV2-5022182L0-05.15

©2015 Brasseler USA. All rights reserved. Made in the U.S.A.

Order Number:

**K0263**

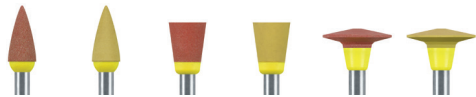


Figure Number	W16MLD	W16FLD	W17MLD	W17FLD	W18MLD	W18FLD
Head Size (1/10 mm)	040	040	060	060	100	100
Head Length (mm)	10.0	10.0	7.5	7.5	2.5	2.5

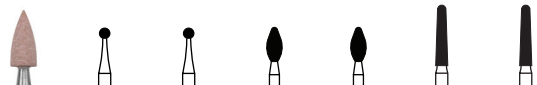








Figure Number	LD11MLD	8801LDF	801LDEF	8369DF	369DEF	8856DF	856DEF
Head Size (1/10 mm)	030	014	014	025	025	016	016
Head Length (mm)	7.5	–	–	5.5	5.5	8.0	8.0








Only Available Direct from your Brasseler Representative  
 To order call 800.841.4522 or fax 888.610.1937  
 In Canada call 800.363.3838 or fax 866.330.4454  
 Visit our website at [BrasselerUSA.com](http://BrasselerUSA.com)



**BRASSELER**  
USA®

## K0263 DIALITE® LD ADJUSTING, FINISHING & POLISHING SYSTEM FOR LITHIUM DISILICATE

Order #	Head Size	Grit	Usage
 W16MLD.RA	040	M	Polisher - Occlusal Surfaces
 W17MLD.RA	060	M	Polisher - Labial/Lingual/Buccal
 W18MLD.RA	100	M	Polisher - Interproximal Surfaces
 W16FLD.RA	040	F	Polisher - Occlusal Surfaces
 W17FLD.RA	060	F	Polisher - Labial/Lingual/Buccal
 W18FLD.RA	100	F	Polisher - Interproximal Surfaces

Order #	Head Size	Grit	Usage
 LD11MLD.RA	030	C	Grinder - Adjusting
 8801LDF-FG	014	F	Endo Access/Small Adjustments
 801LDEF-FG	014	EF	Endo Access/Small Adjustments
 8369DF-FG	025	F	Occlusal Surfaces
 369DEF-FG	025	EF	Occlusal Surfaces
 8856DF-FG	016	F	Contact Surfaces/Crown Removal
 856DEF-FG	016	EF	Contact Surfaces/Crown Removal

### TIPS & TECHNIQUES:

Polishers produce best results when using both grits in order (medium and fine); medium red for pre-polishing and fine yellow for final polishing. Any adjustments should be made with the LD Grinder medium pink or Dialite Diamonds.

Complimentary items:

Intra-Oral LD Grinders (LD11M.RA or LD11C.RA) and Dialite Diamonds (Round, Football and Round-End Taper)

Power Suggestion:

Brasseler Forza F1 Electric Attachment with light pressure

Sterilization:

Autoclave only

Recommended Speeds:

5,000-7,000 rpms



**BRASSLER**  
USA®